




Definisse™ Threads

RE*shape techniques*

RELIFE
MENARINI group



The whole is more than the sum of its parts
Aristotle (384-322 BC)

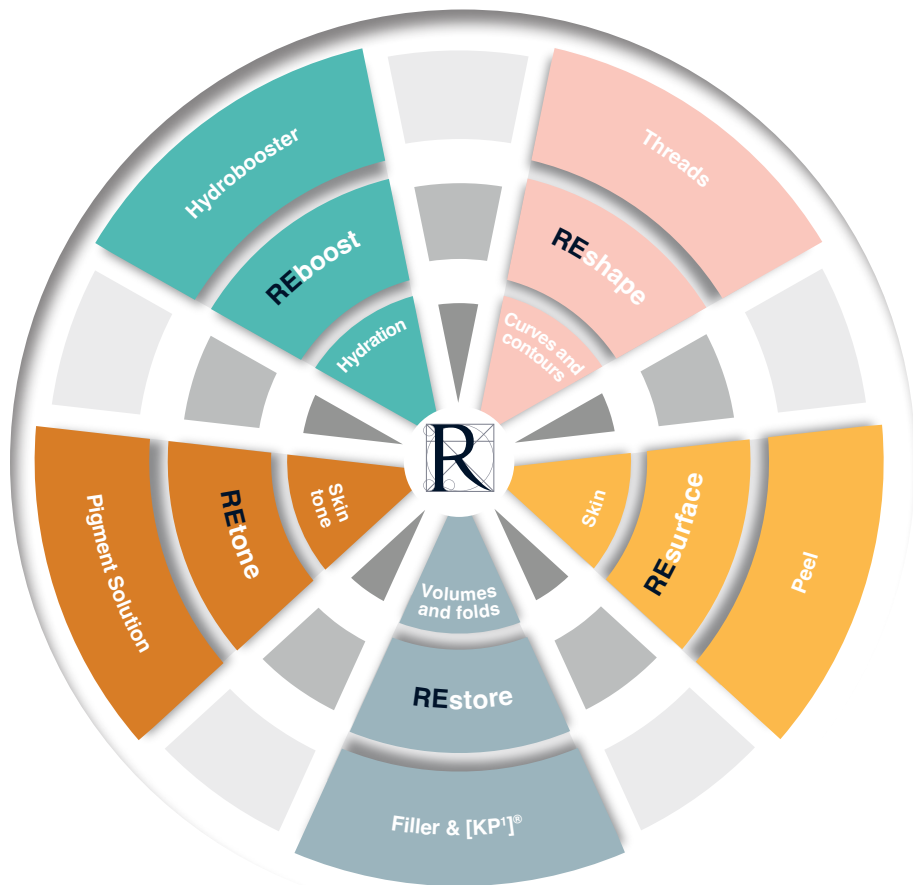

5R

ESSENTIALS

MILLENNIALS PREJUVENATION

The latest cosmetic trends in the millennial consumer group is prejuvenation (prevention and rejuvenation).¹


The prejuvenation treatment with combination aesthetic modalities has risen in popularity amongst millennials.¹

2x aesthetic procedures nearly doubled in the under 30 years old, in the last 6 years.¹

82% millennial consumers believe that injectable treatment is socially acceptable.¹

Definisse™ double needle threads

Available techniques: JR, MR, LR, OR-V, OR-H, ERT

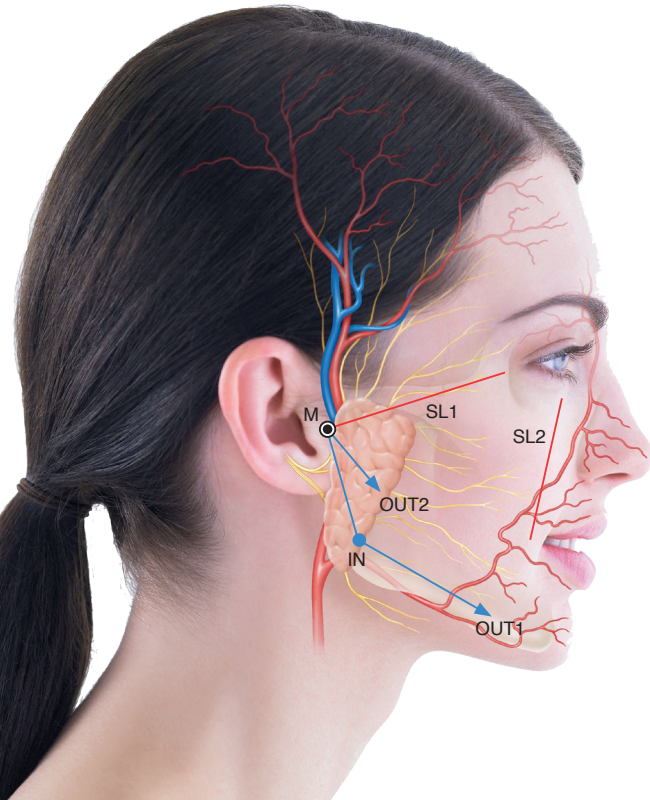

JR technique

- The Definisse™ 12 cm double needle threads, used with the **Jawline Reshaping (JR) Technique**, allow to reshape the frame of the jawline, providing a minimally invasive thread lifting of the lower face.

The Definisse™ double needle threads
are bidirectional and convergent threads


OUTCOME



Photos courtesy of Dr. S. Fundarò

- Lifting action of the inferior jaw fat compartment
- Sharpened mandibular angle
- Repositioning of the pre-auricular area soft tissues

Definisse™ double needle threads

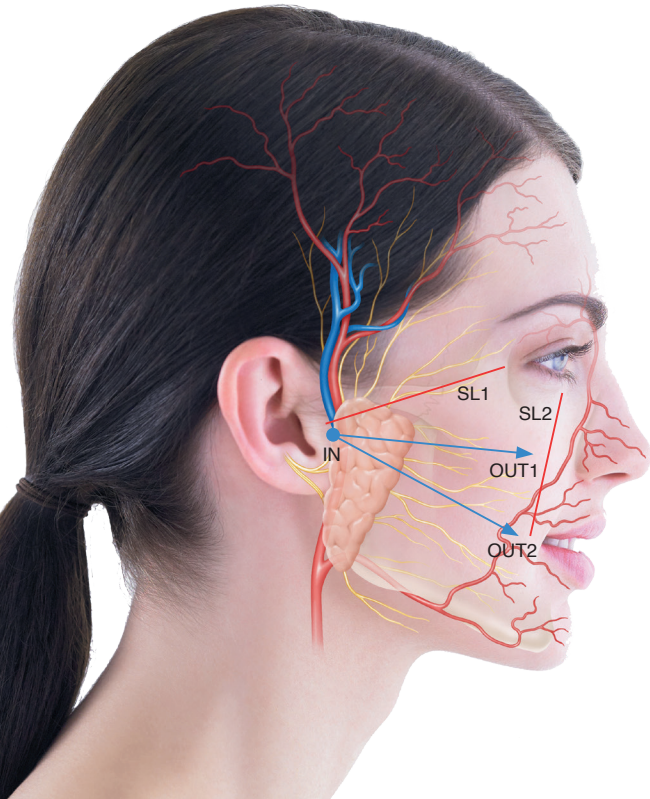

MR technique

- The Definisse™ 12 and 23 cm double needle threads, used with the **Malar Reshaping (MR) Technique**, allow to reshape the facial frame, lifting in an upper and more lateral position the superficial fat compartments of malar and cheekbone area.


OUTCOME



Photos courtesy of Dr. S. Fundarò

- Repositioning of the malar area soft tissues
- Lifting action of the nasolabial folds

Definisse™ double needle threads

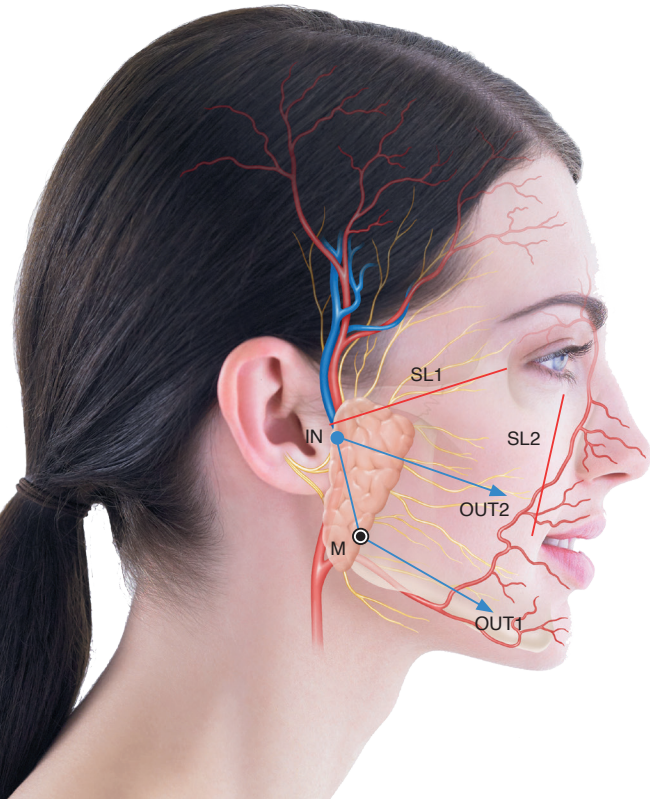

LR technique

- The Definisse™ 23 cm double needle threads, used with the **Lateral Reshaping (LR) Technique**, allow to reshape the facial frame, lifting in an upper and more lateral position the superficial fat compartment of the cheek.
- This technique is more indicated in Caucasian patients.


OUTCOME



Photos courtesy of Dr. S. Fundarò

- Repositioning of the nasolabial fat compartment in a more lateral and upper position
- Lifting action of the inferior jaw fat compartment

Definisse™ double needle threads

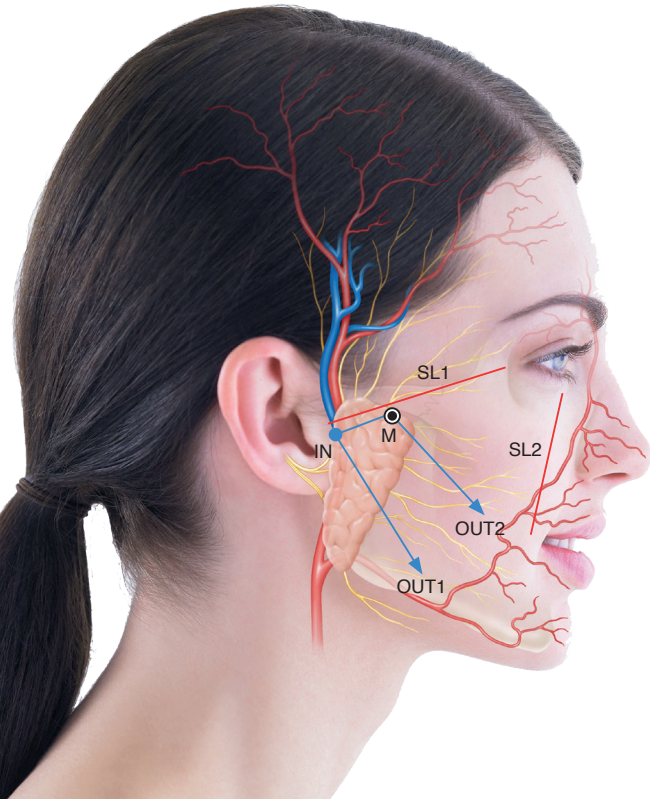

OR-V technique

- The Definisse™ 23 cm double needle threads, used with the **Oval Reshaping (OR-V) Technique**, allow to reshape the facial frame, making the face more oval and lifting the mid- and lower-face superficial cheek fat compartment.


OUTCOME



Photos courtesy of Dr. P. Danai

- Ovalisation of the facial frame
- Lifting action of the nasolabial folds
- Lifting action of the mouth corners


Definisse™ double needle threads

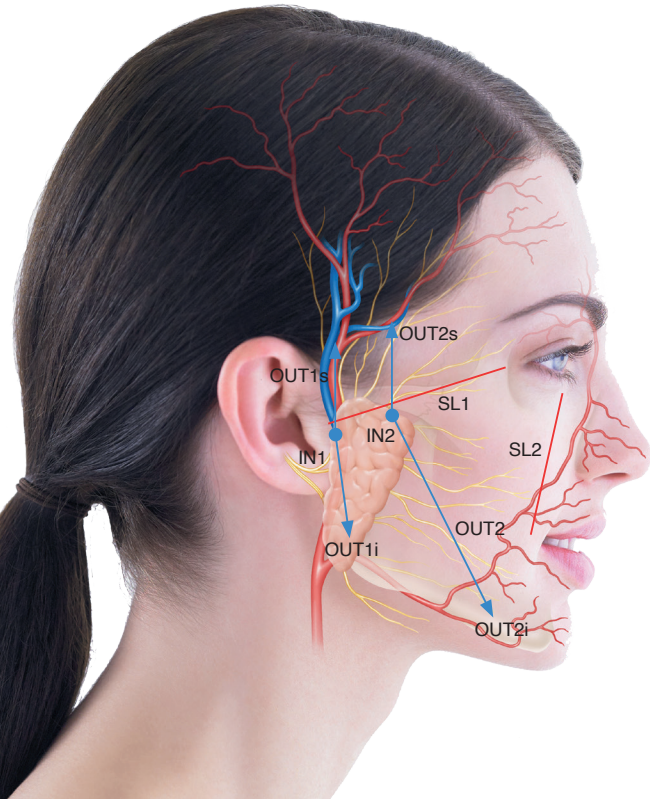

OR-H technique

- The Definisse™ 12 cm double needle threads, used with the **Oval Reshaping (OR-H) Technique**, allow to vertically reshape the frame of the face, providing a non invasive thread lifting.


OUTCOME

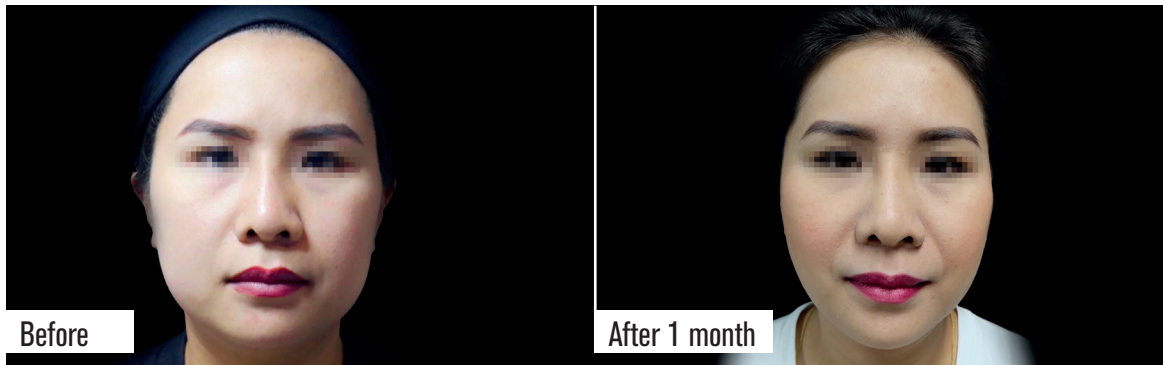

Photos courtesy of Dr. P. Danai

- Verticalisation of the facial frame
- Lifting action of the nasolabial folds
- Lifting action of the corners of the mouth


Definisse™ double needle threads

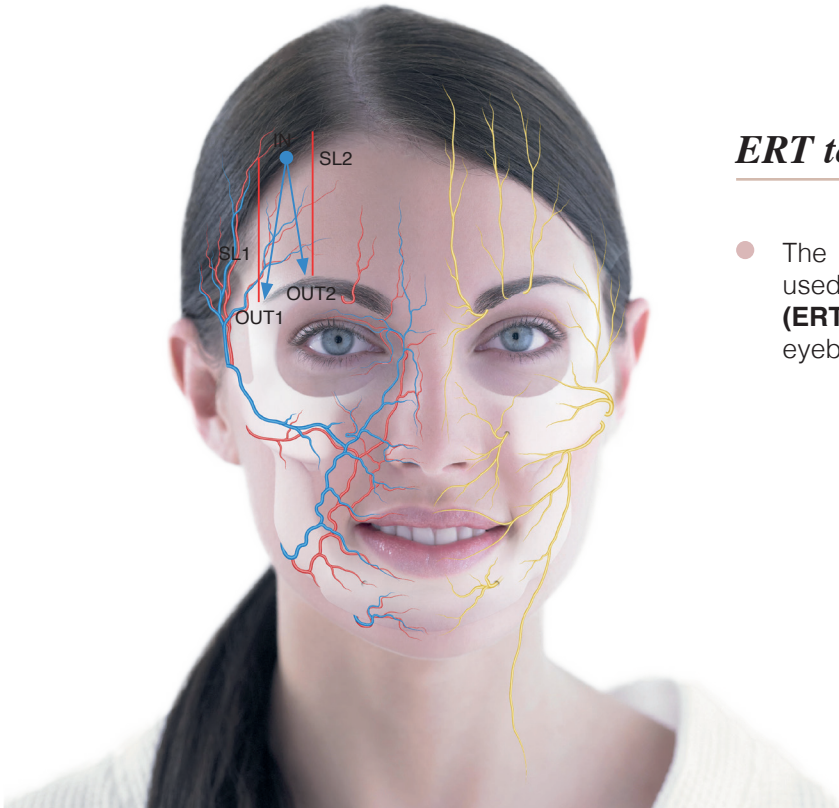

ERT technique

- The Definisse™ 12 cm double needle threads, used with the **Eyebrow Reshaping Technique (ERT)**, allows to widen the eyes, rising the eyebrow's tail.

OUTCOME

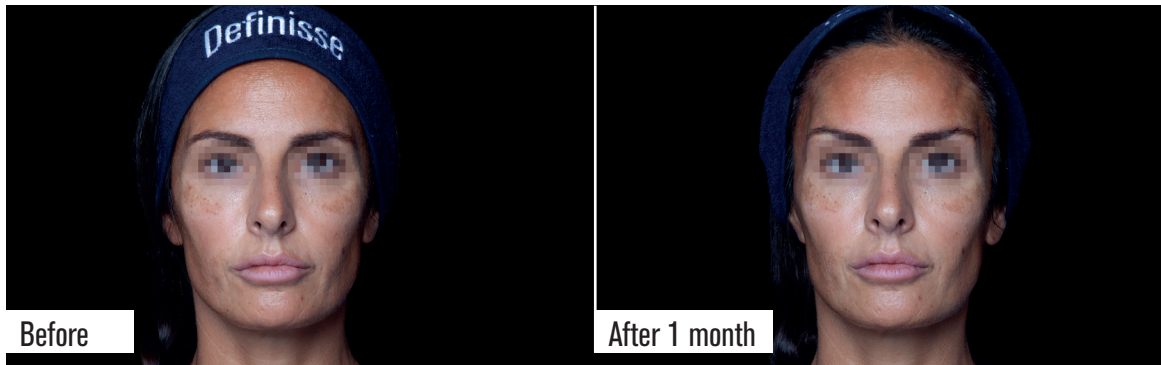


Photos courtesy of Dr. A. Santorelli

- Reshaping of the eyebrow curve
- Widening the eyes ("Foxy eye effect")

Definisse™ free floating threads

Available techniques: STR-2, STR-3

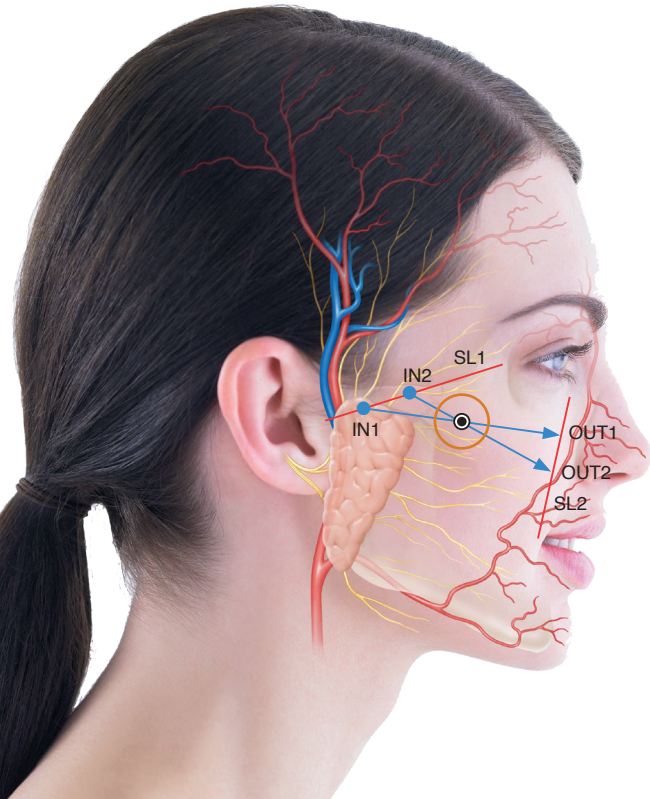

STR-2 technique

- The Definisse™ 12 cm free floating threads, used with the **Soft Tissue Reshaping-2 (STR-2) Technique**, allow to reposition the tissues of the malar area in subjects affected by mild ptosis.

The Definisse™ free floating threads are bidirectional and convergent threads


OUTCOME



Photos courtesy of Dr. S. Vourtsis

- Volumising action of the cheekbone area
- Lifting action of the nasolabial folds

Definisse™ free floating threads

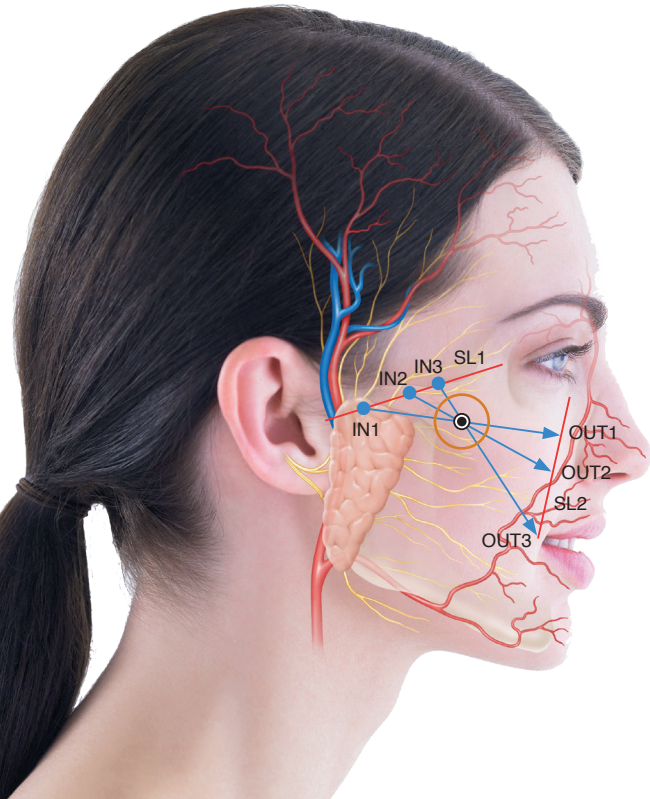

STR-3 technique

- The Definisse™ 12 cm free floating threads, used with the **Soft Tissue Reshaping-3 (STR-3) Technique**, allow to reposition the tissues of the malar area in subjects affected by moderate ptosis.

OUTCOME




Photos courtesy of Dr. S. Vourtsis

- Volumising action of the cheekbone area
- Lifting action of the nasolabial folds

Definisse™ ancorage threads

Available techniques: AGR-J, AGR-M

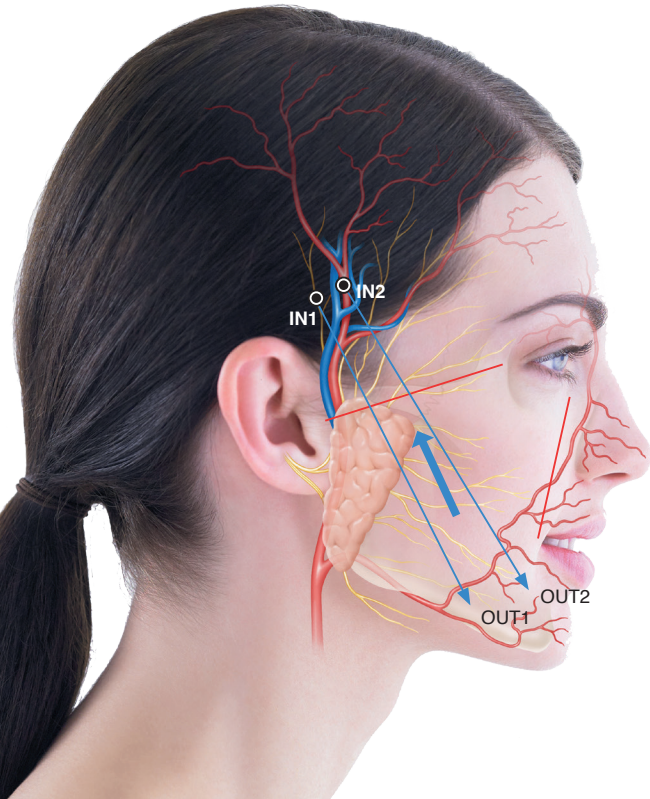

AGR-J technique

- The Definisse™ ancorage threads, used with the **Antigravity Jawline Reshaping (AGR-J) Technique**, allow to reshape the lower face unit, providing a minimally invasive thread lifting.
- This is an **Advanced** technique.

The Definisse™ ancorage threads
are unidirectional threads


OUTCOME



Photos courtesy of Dr. P. Cirillo

- Reduction of the facial jowling
- Lifting action of the prejaw sulcus and the marionette lines
- Lifting action of the jaw and malar area

Definisse™ ancorage threads

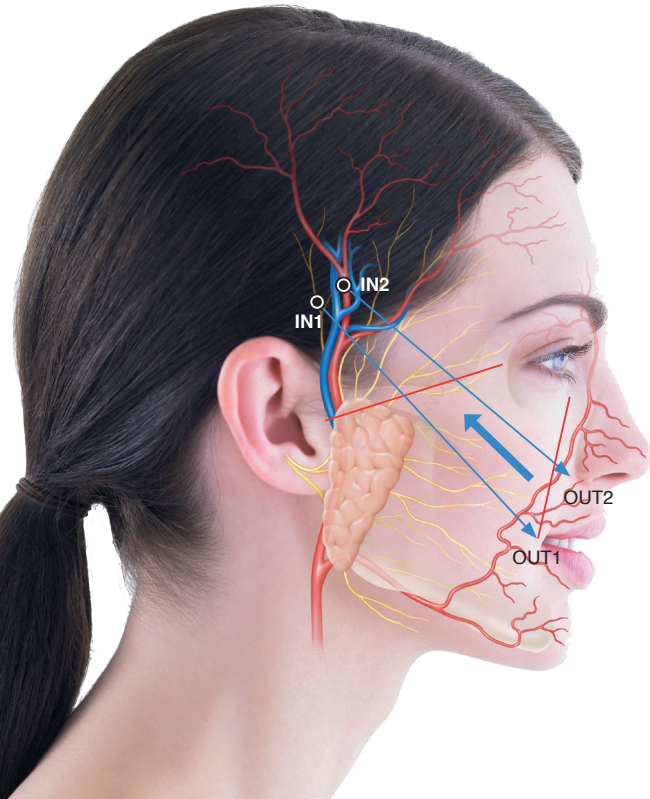

AGR-M technique

- The Definisse™ ancorage threads, used with the **Antigravity Malar Reshaping (AGR-M) Technique**, allow to reshape the all midface unit, providing a minimally invasive thread lifting.
- This is an **Advanced** technique.


OUTCOME



Photos courtesy of Dr. P. Cirillo

- Repositioning of the malar area
- Lifting action of the nasolabial folds


Definisse™ boca threads

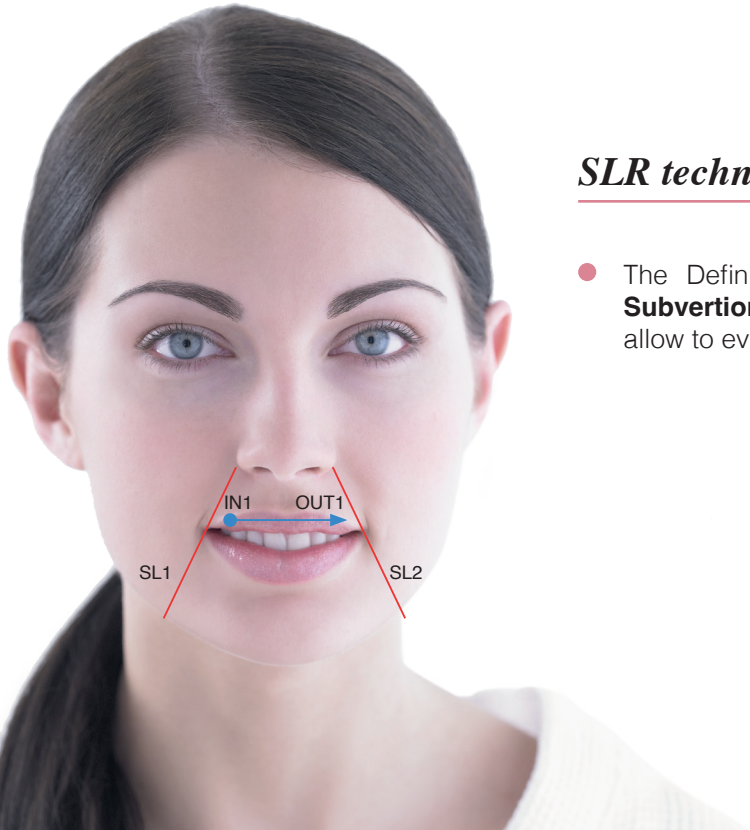

SLR technique

- The Definisse™ boca threads, used with the **Subversion Lip Reshaping (SLR) Technique**, allow to evertering lips.


OUTCOME

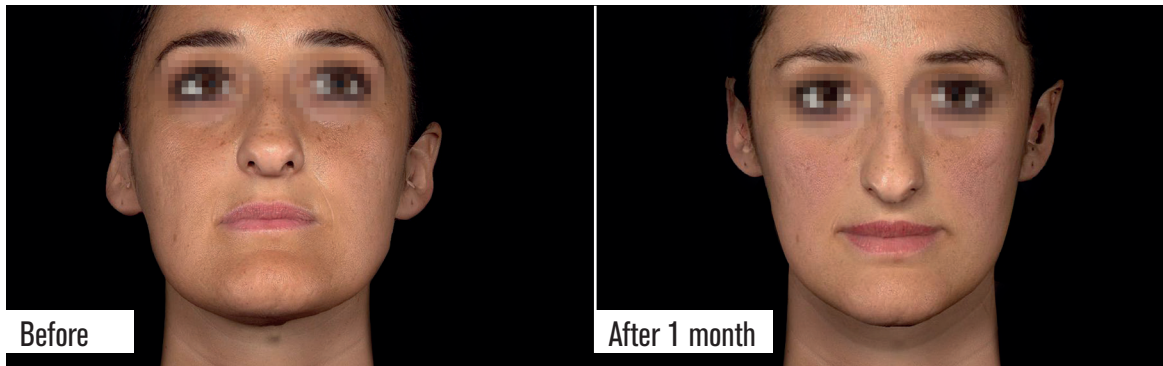

Photos courtesy of Dr. A. Santorelli

- Subversion of the lips
- Augmentation of the lip volume
- Wider natural-looking lip shaping

Definisse™ nose threads

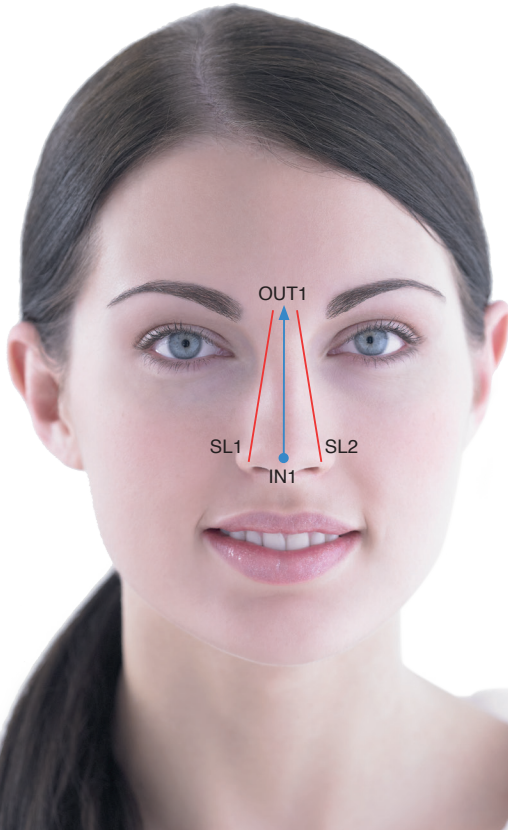

NTR-1 technique

- The Definisse™ nose threads with the **Nose Tip Reshaping (NTR-1) Technique**, allow to elevate the nose tip projection in subjects affected by a mild ptosis.


OUTCOME



Photos courtesy of Dr. A. Santorelli

- Raising of the tip nose
- Natural-looking effect without adding volume

Definisse™ nose threads

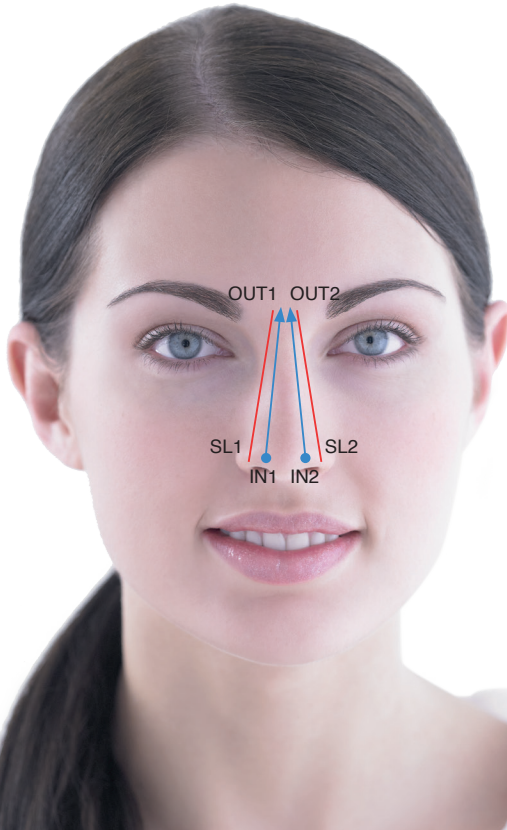

NTR-2 technique

- The Definisse™ nose threads with the **Nose Tip Reshaping (NTR-2) Technique**, allow to elevate the nose tip projection in subjects affected by a moderate ptosis.

OUTCOME



Photos courtesy of Dr. A. Santorelli

- Raising of the tip nose
- Natural-looking effect without adding volume

Threads	Technique	Indications	Actions
Definisse™ double needle	Jawline Reshaping (JR)	<ul style="list-style-type: none"> Lifting action of the jawline and pre-auricular area Definition of the jaw angle 	<input type="checkbox"/> IMMEDIATE ACTION: volumising + lifting action ++ <input type="checkbox"/> LONG TERM ACTION: revitalisation due to the fibrosis
	Malar Reshaping (MR)	<ul style="list-style-type: none"> Lifting action of the malar area Lifting action of the nasolabial fold 	
	Lateral Reshaping (LR)	<ul style="list-style-type: none"> Lifting action of the nasolabial fold Lifting action of the jawline and the supero-lateral neck 	
	Oval Reshaping (OR-V)	<ul style="list-style-type: none"> Lifting action of the jawline and the mouth angle 	
	Oval Reshaping (OR-H)	<ul style="list-style-type: none"> Vertically reshaping the frame of the face 	
	Eyebrow Reshaping (ERT)	<ul style="list-style-type: none"> Reshaping of the eyebrow curve 	
Definisse™ free floating	Soft Tissue Reshaping-2 (STR-2)	<ul style="list-style-type: none"> Volumising action of the cheekbone Lifting action of the upper nasolabial fold and the cheekbone MILD PTOSIS	<input type="checkbox"/> IMMEDIATE ACTION: volumising ++ mild lifting action + <input type="checkbox"/> LONG TERM ACTION: revitalisation due to the fibrosis
	Soft Tissue Reshaping-3 (STR-3)	<ul style="list-style-type: none"> Volumising action of the cheekbone Lifting action of the nasolabial fold and the cheekbone MODERATE PTOSIS	
Definisse™ ancorage	Antigravity Jawline Reshaping (AGR-J)	<ul style="list-style-type: none"> Lifting action of the wattle and the marionette line Lifting action of the jawline and malar area 	<input type="checkbox"/> IMMEDIATE ACTION: mild volumising + lifting action +++ <input type="checkbox"/> LONG TERM ACTION: revitalisation due to the fibrosis
	Antigravity Malar Reshaping (AGR-M)	<ul style="list-style-type: none"> Lifting action of the malar area and the cheekbone Lifting action of the nasolabial fold 	

Threads	Technique	Indications	Actions
Definisse™ boca	Subversion Lip Reshaping (SLR)	<ul style="list-style-type: none"> • Subversion of the lips • Volumising action of the lips 	<input type="checkbox"/> IMMEDIATE ACTION: volumising action + subversion action ++ <input type="checkbox"/> LONG TERM ACTION: revitalisation due to the fibrosis
Definisse™ nose	Nose Tip Reshaping (NTR-1)	<ul style="list-style-type: none"> • Raising of the nose tip for a mild ptosis 	<input type="checkbox"/> IMMEDIATE ACTION: lifting action ++
	Nose Tip Reshaping (NTR-2)	<ul style="list-style-type: none"> • Raising of the nose tip for a moderate ptosis 	<input type="checkbox"/> LONG TERM ACTION: revitalisation due to the fibrosis

FOR FACIAL CURVES AND CONTOURS RESHAPING

- 1 The peculiarity of **Definisse™ Threads** is the presence of a special kind of barb, created by a unique technique with the aim of not reducing the gauge of thread^{2,5}
- 2 **Definisse™ Threads** are a range of absorbable, monofilament, suspension-barbed threads of synthetic origin poly (L-lactide-co-ε-caprolactone, p(LA-CL))^{2,5}
- 3 The **Definisse™ Threads** interact with the tissues with a double action:
 - Lifting action for the support and repositioning of the tissues, thanks to the mechanical action^{2,5}
 - Revitalising action stimulating synthesis of collagen, hyaluronic acid and elastin, thanks to the barbs and the specific material^{2,5}
- 4 Clinical studies show that **Definisse™ Threads** allow:
 - **An immediate** facial reshaping^{3,5}
 - **A good satisfaction** for the patients^{4,5}

References

1. Mobayed N., Nguyen J.K., Jagdeo J., Minimally Invasive Facial Cosmetic Procedures for the Millennial Aesthetic Patient, JDD 2020 Millennials.
2. Berardesca E. Reshaping with barbed threads. *Minerva Medica*. Feb 2020.
3. Savoia A. et al. Outcomes in thread lift for facial rejuvenation: a study performed with Happy Lift™ revitalizing. *Dermatol Ther* 2014; 4: 103-114.
4. Kalyan R. et al. The Efficacy of Polycaprolactone Threads in Zygomatic and Mandibular Lifting: consecutive Study from a Single Practitioner's Experience. *Int J Clin Exp Dermatol* 2017; 1(1): 1-3.
5. V.Wong, "The Science of Absorbable Poly(L-Lactide-Co-ε-Caprolactone) Threads for Soft Tissue Repositioning of the Face: An Evidence-Based Evaluation of Their Physical Properties and Clinical Application", *Clinical, Cosmetic and Investigational Dermatology*, 2021.



Conscious Innovation in Well-Being

It is time for a new approach that considers the skin as the mirror of a complete well-being. RELIFE is a long-term project focused on helping people to achieve a complete well-being, starting from their skin. RELIFE partners with Specialists and Scientific Societies from around the world to provide consumers with high quality products based on scientific innovation.

This is why at RELIFE we talk about conscious innovation: premium, science-based innovation at the service of our consumers.



Discover more at www.relife-icme.com



RELIFE Srl Via Dei Sette Santi 3 - 50131 Florence, ITALY
Tel. +39 055 56.80.93.45 - Fax +39 055 56.80.93.46

Definisse™ Threads: CE0373 (ASSUT EUROPE SPA)

The information in this material is intended for Healthcare Professional only
R18010/2